



SILVERQUINE[®]

Animal Wound Dressing Gel

CHANGING THE DRESSING

1. Carefully remove the secondary dressing from the wound.
2. Residual SILVERQUINE[®] may be removed with the cover dressing.
3. Follow instructions above to re-apply the gel.

FREQUENCY OF CHANGE

SILVERQUINE[®] should normally be changed every 24 to 72 hours, but the frequency will depend on the type and condition of the wound.

PRECAUTIONS

1. Consult with a healthcare professional if you have any concerns or questions about the wound.
2. Be alert for signs of wound infection. These can include:
 - Pus or cloudy fluid draining from the wound
 - Increased redness around the wound
 - Wound has become extremely tender
 - Increased pain or swelling 48 hours after the wound occurred
 - Onset of fever
 - Wound has not healed within 10 days from the date of injury
3. SILVERQUINE[®] is designed for use on any part of the body except for oral and ophthalmic use. Consult a healthcare professional before using the gel in any other manner.

STORAGE

Store at room temperature.

MADE IN THE USA

Distributed by: **Silverquine LLC**
4839 Volunteer Road, SW Ranches, FL 33330
www.silverquine.com

Questions & Comments:
1-800-317-5917

RO - 7/2/2014



SILVERQUINE[®]

Animal Wound Dressing Gel

PRODUCT DESCRIPTION

SILVERQUINE[®] is a water-base hydro gel wound dressing for use in moist wound care management.

SILVERQUINE[®] contains silver that may help inhibit the growth of microorganisms within the dressing.

SILVERQUINE[®] **Animal Wound Dressing Gel** has been evaluated in standard tests that show it can reduce the level of common microorganisms including:

- Staphylococcus aureus
- Pseudomonas aeruginosa
- Escherichia coli
- MRSA and VRE
- As well as fungi such as Candida albicans and Aspergillus niger. (USP <51>).

INDICATIONS FOR USE

- For topical management of:
- Minor Cuts
 - Lacerations
 - Abrasions
 - 1st and 2nd Degree Burns
 - Skin Irritations

DIRECTIONS FOR USE

1. Wash your hands with soap and water before touching the wound.
2. Carefully clean the wound using soap and water or an appropriate commercial brand of wound cleanser.
3. Dispense SILVERQUINE[®] onto an appropriate clean applicator (such as a tongue depressor or gauze) in a sufficient amount to liberally cover the wound. Cover the gel with an appropriate secondary dressing cover such as a gauze bolster, dressing wrap, film or non-woven adhesive secondary dressing.

MADE IN THE USA

Distirbuted by: **Silverquine LLC**
4839 Volunteer Road, SW Ranches, FL 33330
www.silverquine.com

Questions & Comments:
1-800-317-5917

RO - 7/2/2014